



Finger on the Trigger

Extend to Use and Retract for Safety


Retractable Safety
Scalpels by

Swann-Morton®



The extended Retractable Scalpel range.


The EU Council Directive 2010/32/EU accepted into UK law under the Health and Safety (Sharps Instruments in Healthcare) regulations 2013 aims to achieve the safest possible environment for Healthcare workers through the prevention of sharps related injuries. Medical devices incorporating engineered safety mechanisms and protection should be "considered" by staff as "part" of their risk assessment process alongside training and education in best practice when handling sharps.

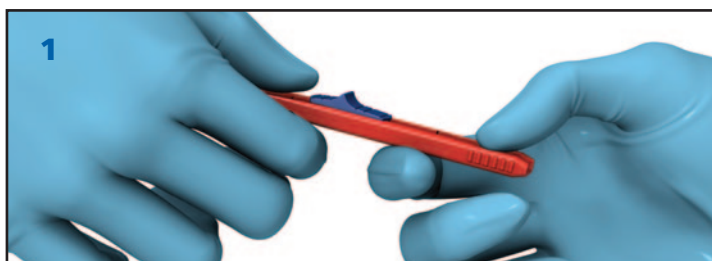

1 The high visibility red handle body contains a mid-safe passing notch into which the coloured slider is located whilst retracted so that the blade is held securely and safely when passing between members of the surgical team.



2 The user can expose the blade by placing downward pressure onto the sliding trigger and pushing it forward to engage the temporary usage position.

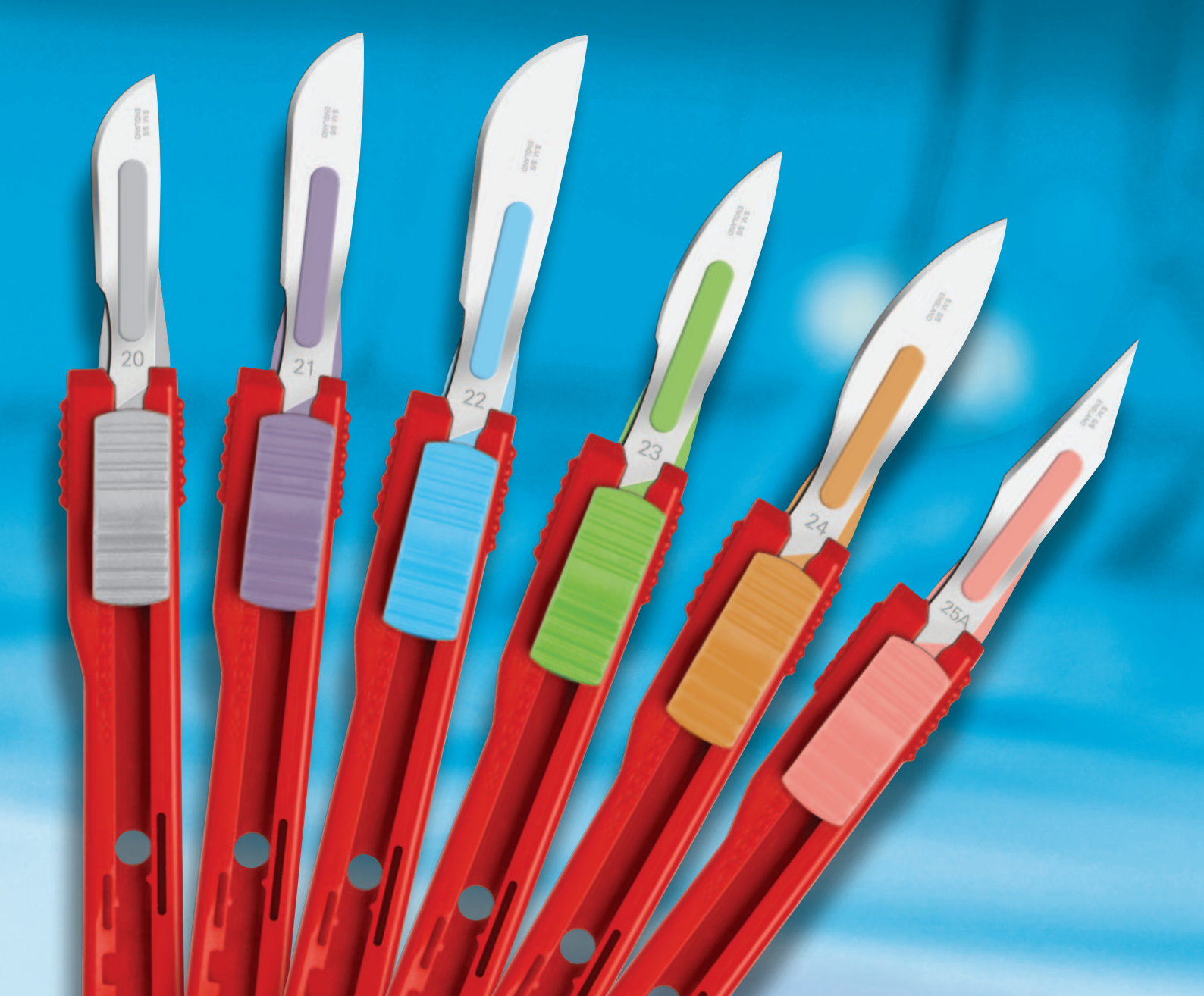


3 This forward lock is strengthened to prevent any uncontrolled retraction of the blade when coming into contact with hard surfaces or bone during use.



4 Once the procedure is finished and the count completed the trigger should be moved from the safe passing position towards the back of the handle body, where with a conscious amount of effort, it will engage the permanent locking position ready for final disposal in a sharps container.



Scan this code for more product information

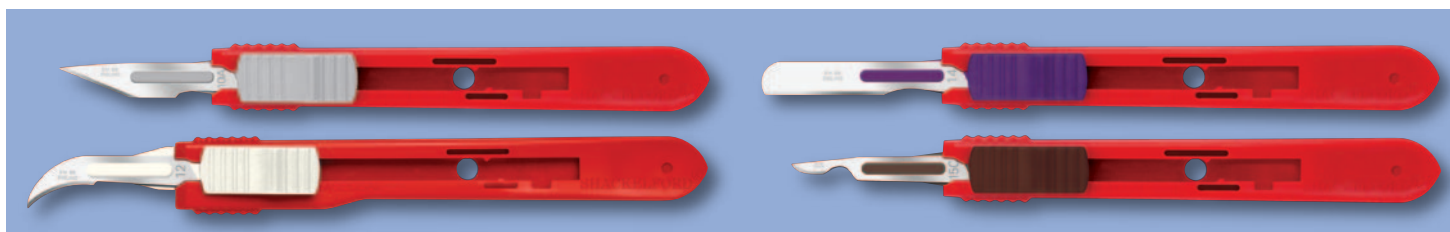

Retractable Disposable Scalpel - Sterile

ABS Handle fitted with Stainless Steel Surgical Blade, individually packed

Product	Product REF	Unit
10 scalpels individually packed		
10	4901	Box of 10
10A	4902	Box of 10
11P	4991	Box of 10
12	4904	Box of 10
14	4919	Box of 10
15	4905	Box of 10
15C	4921	Box of 10
20	4906	Box of 10
21	4907	Box of 10
22	4908	Box of 10
23	4910	Box of 10
24	4911	Box of 10
25A	4915	Box of 10
Stitch Cutter	4926	Box of 10
10R	4990	Box of 10

Product	Product REF	Unit
25 scalpels individually packed		
10	3901	Box of 25
10A	3902	Box of 25
11P	3991	Box of 25
12	3904	Box of 25
14	3919	Box of 25
15	3905	Box of 25
15C	3921	Box of 25
20	3906	Box of 25
21	3907	Box of 25
22	3908	Box of 25
23	3910	Box of 25
24	3911	Box of 25
25A	3915	Box of 25
Stitch Cutter	3926	Box of 25
10R	3990	Box of 25

Retractable Disposable Scalpel - Non-Sterile

ABS Handle fitted with Stainless Steel Surgical Blade, bulk packed

Product	Product REF	Unit
500 scalpels bulk packed		
10	4001	Box of 500
10A	4002	Box of 500
11P	4091	Box of 500
12	4004	Box of 500
14	4019	Box of 500
15	4005	Box of 500
15C	4021	Box of 500
20	4006	Box of 500

Product	Product REF	Unit
500 scalpels bulk packed		
21	4007	Box of 500
22	4008	Box of 500
23	4010	Box of 500
24	4011	Box of 500
25A	4015	Box of 500
Stitch Cutter	4026	Box of 500
10R	4090	Box of 500

Swann-Morton and the 'ring pattern' logo are registered trademarks of Swann-Morton Limited and related companies. The products shown are manufactured for use by trained health care professionals and can cause injury if used or handled incorrectly. The information and illustrations used in connection with the products shown on this leaflet are for general guidance only and users must seek proper training and instruction in their use before using or handling such products. Swann-Morton Limited exclude all liability for injury to the extent permitted by law.